INSTRUCTIONS FOR USE

Hydroven 1 / Hydroven 3

Arm, Leg and Insert Garments

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Instructions for Use · Mode d'emploi · Gebrauchsanleitung · Istruzioni per l'uso · Instrucciones de uso ·
Instruções de uso · Gebruiksaanwijzing · Brugervejledning · Bruksanvisning · Instrukcja użycia · Οδηγίες χρήσης ·
Kullanım için talimatta · Bruksanvisning · Käyttöohjeet ·

Arm, Leg and insert Garments · Atelles de jambe et de bras · Arm- und Beinmanschetten · Tutori per braccia e
gambe · Prendas para brazo y pierna · Acessórios para braços (luva) e pernas (meia) · Arm- en beenmanchetten
· Arm- og bentekstiler · Arm- och benmanschetter · Mankiety na rękę i nogę · Ενδύματα Βραχίονα και Κνήμης ·
Kol ve Bacak Manşonları · Arm og Lår plagg · Arm ja Leg Vaatteet ·

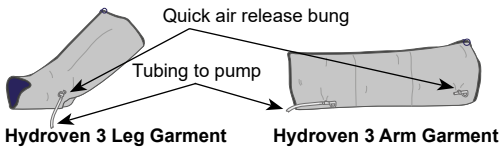
⚠ To be used only under the direction of a physician.

For use with **Huntleigh Hydroven® 3** pump only. Refer to the **Hydroven 3** User Manual for complete information on using the pump and garments. 100% Latex Free.


Before fitting the garment ensure all quick air release bungs are closed, as this will effect the efficiency of the garment.

Applying the Garment

1. Check the type of garment - Hydroven 1 (single chamber) or Hydroven 3 (three chamber) - and the length of the garment required.
2. Undo the zip on the garment.
3. If a garment insert is fitted to the garment, fully fasten one of the zips between the garment and insert, leaving the other unfastened.
4. Before applying the garment (and insert, if fitted) to the limb, zip up the first 150 mm (6") of the unfastened garment zip. Put the garment (and insert) onto the limb and fully fasten the zip. Make sure that the quick air release bung is secured.

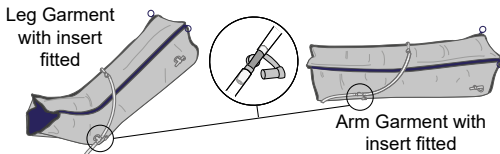5. Make sure the patient is in a comfortable position with the limb supported or elevated as necessary.
6. Check that the insert piece connecting tube is not kinked and is attached to the garment using the lower outlet bung as shown below.

7. Attach the garment tubing to the pump ensuring a "click" is heard from each snap-lock connector.
8. If only one garment is to be used, attach the garment to either port on the pump. The system will automatically identify that only one garment is to be used.
9. Switch on the pump and adjust the pressure control accordingly (refer to the pump user manual).

⚠ WARNINGS

- Do not apply the garment to the limb unless it is partially zipped, as you may damage the garment zip.
- Do not undo the zips or attempt to remove the garments during the therapy session or you may damage the zips. Make sure the therapy session has stopped and the garments have deflated before you remove the garments.
- Do not stand or walk with the garments fitted.
- In the event of a power failure or fault whereby the garment remains inflated, disconnect the tubeset(s) in order to deflate the garment(s) and then remove the garment(s) from the limb(s).

Cleaning the Garment

	Wipe down using a neutral detergent or soap powder at 51°C (120°F).		
	Do not iron		Do not dry clean
	Do not tumble dry	Do not machine wash. Air dry thoroughly.	

Disinfecting the Garment

	After cleaning, wipe complete garment over using 70% isopropyl alcohol wipe or a chlorine-releasing agent at 1000 p.p.m. available chlorine.
	Do not use phenol or phenol-derivative disinfectant.

Rinse with clean water to remove any residue. Air dry thoroughly.

Cleaning the Tubeset

Use a soft brush. Air dry only. Do not immerse in water. Do not machine wash.

Note

- Always refer to local protocols and guidelines, as some protocols recommend single patient use outside of a clinically controlled environment, to avoid cross contamination.
- Garments are designed to be worn over thin clothing and not in direct contact with the patients skin.

This section is only applicable to United Kingdom (UK) market when UK marking is applied to the Arjo medical device labelling.

UK Symbol:

UK marking indicating conformity with UK Medical Devices Regulations 2002 (SI 2002 No 618, as amended) Figures indicate UK Approval Body supervision.

UK Responsible Person & UK Importer:
Arjo (UK) Ltd., Arjo/Huntleigh House, Houghton Regis, LU5 5XF
Is the appointed UK Responsible Person as defined in UK Medical Devices Regulations 2002 (SI 2002 No 618, as amended).

For Northern Ireland (NI) CE marking will still apply until further amendment to applicable regulations.

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©Huntleigh Healthcare Limited 2015
Manufactured for **Huntleigh Healthcare Ltd** on behalf of;
ArjoHuntleigh AB
Huntleigh Healthcare Ltd.